

signia

Life sounds brilliant.

New!

Signia Xperience

**HEAR WHAT MATTERS
TO YOU.**



Signia.

Taking the future of a sound legacy to the next level.


We invent the future of better hearing. Beginning with Siemens, this has been our consistent mission for 140 years and counting – the greatest legacy in the industry. As hearing aid pioneers, we have introduced countless world’s firsts and now we are continuing this success story by introducing our game-changing platform Signia Xperience and enhancing the world of hearing aids.

Hearing solutions for everyone.

With Signia hearing aids, you can offer your customers a wide selection from many different models and performance levels for every requirement and all degrees of hearing loss. To facilitate orientation and help in the selection of the right hearing solution, our hearing aids are placed in three categories according to their construction models. This lets you find out quickly which model best suits a customer’s wearing preferences.

Contents

Hearing solutions for everyone	2	Cellion primax	29
Hearing aids overview	4	Carat primax	30
Signia Xperience feature overview	5	Motion primax	31
Signia Nx feature overview	6	Silk primax	32
primax feature overview	7	Insio primax	33
Essential segment feature overview	8	Intuis 3 RIC312	34
Lotus segment feature overview	9	Intuis 3 BTEs	35
What’s New	10	Intuis 3 Click ITEs and ITEs	36
Pure Charge&Go X	14	Prompt	37
Inductive Charger II	15	Run	38
Pure 312 X	16	Fun	39
Styletto Connect	17	Fast	40
Styletto	18	SecureEar	41
Pure Charge&Go Nx	19	Signia app overview	42
Pure 312 Nx	20	Signia app feature overview	43
Pure 13 Nx	21	Apps, accessories and hearing aid compatibility	44
Pure 10 Nx	22	Accessory overview	45
Motion Charge&Go Nx	23	Color overview	46
Motion 13 Nx	24	Hearing care professional support	48
Silk Nx	25	Overview SLIM-RIC/RICs	50
Insio Nx	26	Overview BTEs	52
Ace primax	27	Overview Click ITEs/ITEs	56
Pure primax	28		

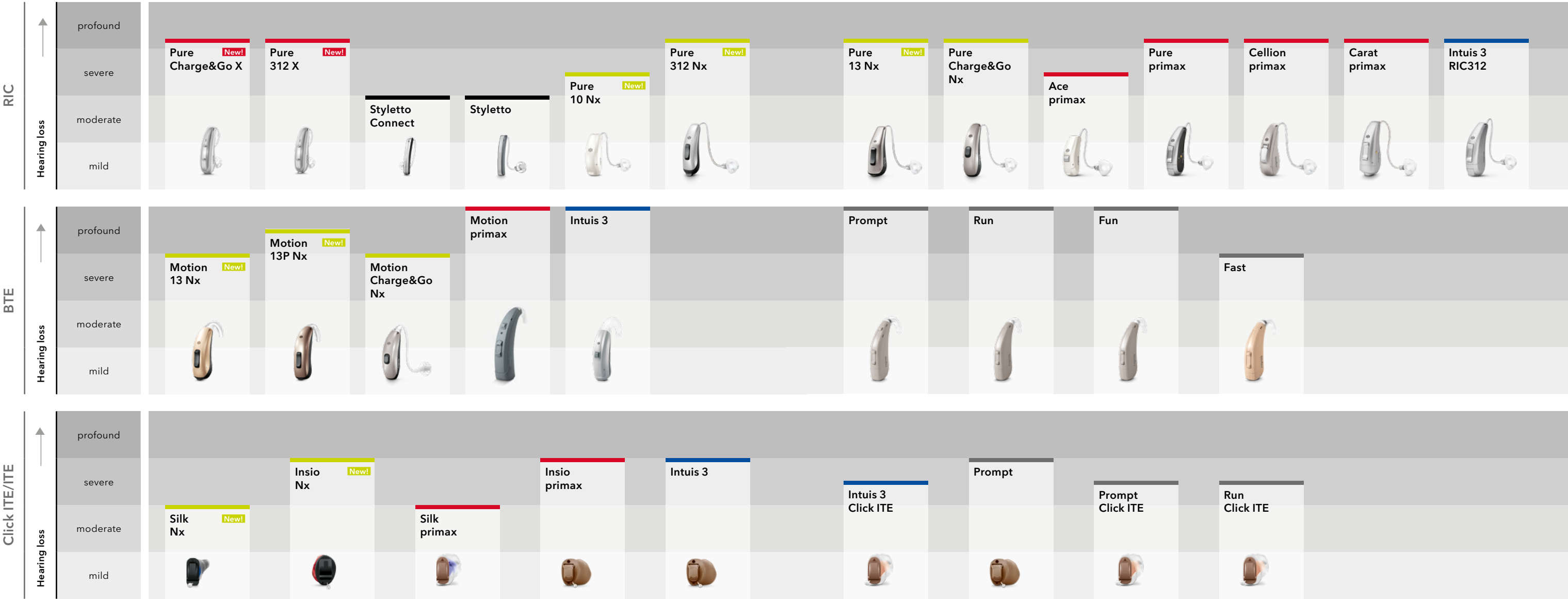

The degrees of hearing loss indicated above serve as a rough guide. For precise information, please refer to the individual product pages.

Hearing aids overview.

We offer a broad range of hearing aids in many different models and performance levels to help hearing care professionals meet their clients' needs, preferences and demands. Take advantage of this diversity to enhance your customers' satisfaction.

Models		Performance levels	RIC		BTE		ITE				Fitting
			Slim	Regular	LifeTube/ ThinTube	Earhook	IIC	CIC	ITC	ITE	Connexx
Xperience	Pure Charge&Go X <small>New!</small>	7X 5X 3X		•							9
	Pure 312 X <small>New!</small>	7X 5X 3X		•							9
Styletto	Styletto Connect	7 5 3	•								9
	Styletto	7 5 3 2 1	•								9
Nx	Pure 10 Nx <small>New!</small>	7Nx 5Nx 3Nx 2Nx 1Nx		•							9
	Pure 312 Nx <small>New!</small>	7Nx 5Nx 3Nx 2Nx 1Nx		•							9
	Pure 13 Nx <small>New!</small>	7Nx 5Nx 3Nx 2Nx 1Nx		•							9
	Pure Charge&Go Nx	7Nx 5Nx 3Nx 2Nx 1Nx		•							9
	Motion 13 Nx <small>New!</small>	7Nx 5Nx 3Nx 2Nx 1Nx			•	•					9
	Motion 13P Nx <small>New!</small>	7Nx 5Nx 3Nx 2Nx 1Nx			•	•					9
	Motion Charge&Go Nx	7Nx 5Nx 3Nx 2Nx 1Nx			•	•					9
	Silk Nx <small>New!</small>	7Nx 5Nx 3Nx 2Nx 1Nx						•*			9
	Insio Nx	7Nx 5Nx 3Nx					•	•	•	•	9
primax	Ace primax	7px 5px 3px 2px 1px		•							9
	Pure primax	7px 5px 3px 2px 1px		•							9
	Cellion primax	7px 5px 3px 2px		•							9
	Carat primax	7px 5px 3px		•							9
	Motion primax	7px 5px 3px 2px 1px			•	•					9
	Silk primax	7px 5px 3px 2px 1px						•*			9
	Insio primax	7px 5px 3px 2px 1px					•	•	•	•	9
Intuis	Intuis 3				•	•		•	•	•	9
	Intuis 3 RIC312			•							9
Lotus	Prompt				•	•		•	•	•	9
	Run				•	•		•*	•*		MobileFitting app
	Fun					•					Manual Fit
	Fast					•					Manual Fit

* Only available as Click ITE.

Signia Xperience feature overview.

Our revolutionary platform Signia Xperience gives hearing aids a new sense to truly understand what the wearer needs to hear even in noise, all combined with the most advanced connectivity systems.

Features	Information	Performance levels		
Audiology		7X	5X	3X
Dynamic Soundscape Processing	Redesigned acoustic sensors plus an integrated motion sensor provide detailed information about where the wearers are and what they are doing. This information is used to steer Sound Clarity and Speech Quality features for natural sound and speech in every situation, even when moving.	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●
OVP™ (Own Voice Processing)	Own Voice Processing (OVP™) utilizes real-time recognition of the wearer's voice to deliver a natural own-voice impression. Available for RIC/BTE devices with e2e wireless.	● ● ● ● ●	● ● ● ● ●	● ● ● ● ●
Sound Clarity	Sound Clarity provides lifelike sound which covers the full spectrum of wearing situations.	● ● ● ● ●	● ● ● ●	● ● ●
Speech Quality	Speech Quality delivers improved speech understanding and communication ease in noise.	● ● ● ● ●	● ● ● ●	● ● ●
User Interaction	Technically inclined wearers may appreciate having extra control over their hearing aids. Signia Xperience hearing aids offer these wearers a comprehensive range of adjustment options via the Signia app.	● ● ● ● ●	● ● ● ● ●	● ● ● ●
Connectivity				
Direct Streaming	Signia Xperience hearing aids can be connected directly to Apple devices for phone calls and audio streaming - without additional accessories. For Android devices the StreamLine Mic accessory makes streaming easy. The StreamLine TV accessory sends high quality stereo sound from the wearer's TV directly to the Xperience hearing aids. Available for Bluetooth enabled fittings.	✓	✓	✓
Remote hearing care				
TeleCare	TeleCare provides remote services including additional tools to follow-up the patients within the whole journey, beginning with pre-appointment and even continuing after purchase. It contributes to a more successful trial phase and ensures higher flexibility. In addition, the user engagement and autonomy during this whole journey will be increased via the Signia app.	✓	✓	✓

● ● ● ● ● = highest feature performance

Signia Nx feature overview.

Our game-changing platform Signia Nx delivers the most natural own voice and speech understanding in noise, while combining the most advanced connectivity systems.

Features	Information	Performance levels				
Audiology		7Nx	5Nx	3Nx	2Nx	1Nx
OVP™ (Own Voice Processing)	Own Voice Processing (OVP™) utilizes real-time recognition of the wearer’s voice to deliver a natural own-voice impression. Available for SLIM-RIC/RIC/BTE devices with Ultra HD e2e.	●●●●●	●●●●●	●●●●●	-	-
3D Classifier	Unlike conventional classification systems which rely on acoustic data, 3D Classifier analyzes multi-dimensional information from the wearer’s environment and communication behavior. Information on the wearer’s voice activity, motion state and acoustic data are used to tailor the hearing aids’ response to the wearer’s needs. Motion detection available for Bluetooth-enabled devices.	●●●●●	●●●●	●●●	-	-
Sound Clarity	Sound Clarity provides lifelike sound which covers the full spectrum of wearing situations. Exceptional clarity is achieved with features such as: HD Spatial, extended dynamic range, extended bandwidth, EchoShield, HD Music, eWindScreen™, speech and noise management and feedback cancellation.	●●●●●	●●●●	●●●	●●	●●
Speech Quality	Speech Quality delivers improved understanding of other people’s voices in noise by using an exclusive technology called Ultra HD e2e. Features which utilize Ultra HD e2e include narrow directionality, Spatial SpeechFocus™ and TwinPhone.	●●●●●	●●●●	●●●	●●	●●
Connectivity						
Direct Streaming	Signia Nx hearing aids can be connected directly to Apple devices for phone calls and audio streaming - without additional accessories. With the StreamLine Mic accessory, direct streaming is possible also from Android devices and other Bluetooth mobile devices. With the StreamLine TV accessory, high-quality stereo sound can be streamed directly to Nx hearing aids. Available for Bluetooth-equipped devices.	●●●●●	●●●●●	●●●●●	●●●●●	●●●●●
You Adjust	Technically inclined wearers often appreciate having extra control over their hearing aids. Signia Nx hearing aids offer these wearers a comprehensive range of adjustment options via Signia app.	●●●●●	●●●●●	●●●●	●●●●	●●●●
Remote hearing care						
TeleCare	TeleCare provides remote services including additional tools to follow-up the patients within the whole journey, beginning with pre-appointment and even continuing after purchase. It contributes to a more successful trial phase and ensures higher flexibility. In addition, the user engagement and autonomy during this whole journey will be increased via the Signia app.	●●●●●	●●●●●	●●●●●	●●●●●	●●●●●

●●●●● = highest feature performance

primax feature overview.

Redefining the ease of listening, the outstanding primax™ star platform offers effortless hearing in every situation throughout the day.*

Features		Information		Performance levels				
Audiology				7px	5px	3px	2px	1px
Signal processing and Gain/MPO	Channels/Handles in Connexx™			48/20	32/16	24/12	16/8	16/8
Hearing programs	Maximum available			6	6	6	4	4
SpeechMaster				●	●	●	●	●
HD Music	Number of preset programs			3	1	1	-	-
TwinPhone ^{A,E}	Req. bilateral fitting & e2e wireless™ 3.0			●	●	●	-	-
EchoShield				●	-	-	-	-
Wireless CROS/BiCROS ^{A,E}	Req. bilateral fitting & e2e wireless 3.0			●	●	●	-	-
Directionality	Resolution in channels			48	32	24	16	16
Binaural OneMic directionality ^B	Req. bilateral fitting & e2e wireless 3.0			●●●●●	-	-	-	-
Narrow directionality ^{A,C}	Req. bilateral fitting & e2e wireless 3.0			●●●●●	●●●●	●●●	-	-
Directional microphones ^C				●●●●●	●●●●	●●●	●●	●
Spatial SpeechFocus ^{A,C}	Req. bilateral fitting & e2e wireless 3.0			●●●●●	-	-	-	-
SpeechFocus ^C				●●●●	●●●●	-	-	-
TruEar™	Req. BTE or RIC hearing aids			●●●●●	●●●●	●●●	-	-
Frequency compression				●	●	●	●	●
Extended bandwidth	Up to 12 kHz			●	-	-	-	-
Feedback cancellation				●	●	●	●	●
eWindScreen binaural ^{A,C,D}	Req. bilateral fitting & e2e wireless 3.0			●	●	-	-	-
eWindScreen	Adjustment steps			3	3	1	1	-
Noise reduction	Resolution in channels/ Combined adjustment steps			48/5	32/5	24/3	16/3	16/1
Speech and noise management	Individual adjustment steps			7	5	3	3	1
SoundSmoothing™	Individual adjustment steps			3	3	1	1	-
Directional speech enhancement ^C	Individual adjustment steps			3	1	-	-	-
Adaptive streaming volume ^A	Streaming only, req. Smart Connect & e2e wireless 3.0			●	-	-	-	-
SoundBrilliance™ ^A	Streaming only, req. Smart Connect & e2e wireless 3.0			●	●	-	-	-
Sound equalizer	Number of classes			6	3	-	-	-
User controls, remote controls, streaming								
Signia app ^F	iOS and Android devices only			●	●	●	●	●
Smart Connect ^A	Req. e2e wireless 3.0			●	●	●	●	●
Spatial Configurator ^{A,C}	Req. bilateral fitting & e2e wireless 3.0			●	●	-	-	-
Span	Req. Smart Connect & Signia app or rocker switch			●	●	-	-	-
Direction	Req. Smart Connect & Signia app			●	●	-	-	-
SoundBalance™				●	●	●	●	●
Fitting								
InSituGram™				●	●	●	●	●
Learning/Data logging	Number of classes			6/6	3/6	1/6	-/6	-/6
Acclimatization manager				●	●	●	●	●
Tinnitus								
Tinnitus therapy								
Static therapy signal (5 presets)	Handles in Connexx			20	16	12	4 ^F	-
Ocean wave therapy signal (4 presets)				●	●	●	-	-
Notch therapy				●	●	●	-	-

A) Not for Ace primax B) Only for hearing aids with one microphone each C) Only for hearing aids with two microphones each
D) Not available in the universal program on 5px E) Not for Motion S primax F) No presets

●●●●● = highest feature performance

* See legal note on final page in this document.

Essential segment feature overview.

Proven technology, outstanding comfort: In a broad range of hearing aid models, the Essential line offers a reliable hearing experience, a high level of wearing comfort and easy handling for almost every customer.

Features

Performance levels

Essential

Better hearing

Intuis 3

Frequency channels/Compression controls	12/6
Number of programs	4
Directional microphones ^A	automatic ^B
Frequency compression	●

Sound comfort

Feedback cancellation	●
Speech and noise management	on/off
eWindScreen	-
e2e wireless 3.0	-
e2e wireless 2.0	-
SoundSmoothing	-

Fitting/Individuality

Data logging	●
Acclimatization manager	-
Tinnitus therapy	-

Audio streamer/Hearing aid control

Signia app	●
Smart Connexx app and Signia app	-

A) Not for CIC B) Only BTE

Lotus segment feature overview.

Available in a wide range of fittings and suitable for all levels of hearing loss from mild to profound, the Lotus range offers a good hearing experience for every customer.

Features	Prompt						
	S	P	SP	Click ITEs	CIC	ITC	ITE
Matrix	LifeTube 45/124 dB Earhook 55/124 dB	70/134 dB	80/140 dB	Click CIC 50/113 dB Click ITC 55/116 dB	65/124 dB ¹	65/124 dB ¹	65/124 dB ¹
Channels/Controls	8/4	8/4	8/4	8/4	8/4	8/4	8/4
Programs	4	4	4	Click CIC: 1 Click ITC: 4	4	4	4
Directional microphones	●	●	●	-	-	-	-
T-coil	●	●	●	-	-	○	○
Audio input	-	●	●	-	-	-	-
Noise reduction	situation based	situation based	situation based	situation based	situation based	situation based	situation based
SoundBalance™	●	●	●	●	●	●	●
Feedback cancellation	●	●	●	●	●	●	●

Features	Run				Fun		Fast
	S	P	SP	Click ITEs	P	SP	P
Matrix	LifeTube 45/124 dB Earhook 55/124 dB	70/132 dB	79/140 dB	Click CIC 50/113 dB Click ITC 55/116 dB	70/132 dB	79/140 dB	62/130 dB
Channels	8	8	8	8	6	6	4
Programs	6 preconfigured sound profiles	6 preconfigured sound profiles	6 preconfigured sound profiles	6 preconfigured sound profiles	4 preconfigured sound profiles	4 preconfigured sound profiles	3 preconfigured sound profiles
Directional microphones	●	●	-	-	-	-	-
T-coil	-	-	-	-	-	-	-
Audio input	-	-	-	-	-	-	-
Noise reduction	situation based	situation based	situation based	situation based	always on	always on	always on
SoundBalance™	●	●	●	●	-	-	-
Feedback cancellation	●	●	●	●	●	●	●

(1) Max. values

Hear what matters to you.



Our new Signia Xperience platform uses the most advanced technology and information to analyze and understand every wearer's environment, hearing situation and hearing preference. This allows for the most intelligent hearing aids that scan and process the soundscape with complete spatial accuracy, delivering sound that's clear and detailed even in the noisiest situations.



A moving experience

Conventional hearing aids listen and make educated guesses as to the type of situation the wearer is in. But they cannot see the full picture. Signia Xperience is the first solution to truly understand how the wearer interacts within his or her environment so they always hear what matters and are fully involved in life.

Signia Xperience produces the perfect personalized sound so that the wearer hears everything at just the right volume with full spatial perception to know where each sound is coming from.

As a result, everything sounds natural and wearers hear what matters to them, wherever they are, wherever they go.



The world's first acoustic-motion sensors

Using powerful acoustic sensors combined with a revolutionary motion sensor linked to sound processing, YourSound technology measures more aspects of sound in real time than ever before, including how the wearer's movement affects their hearing.

Motion sensor

In a world's first, this is built into the hearing aids to provide crucial information about the wearer's movement through his or her acoustic surroundings. Based on this data, the sound processing can be optimized to deliver sound as nature intended, whether running in the park or sitting in a meeting.

Acoustic sensors

Sound localization, own voice detection, signal-to-noise ratio, and much more: By collecting more information and characteristics about the hearing aid wearer's acoustic environment than ever before, the soundscape is brought to life far more precisely and in much greater detail.



Mastering the perfect personalized sound

Drawing on the unique acoustic-motion sensors' precise input, YourSound adapts to wearers' changing personal needs as they move throughout their day. It always understands where they are and what they are doing.

Dynamic Soundscape Processing

Delivers natural sound and speech in every situation even when moving.

Own Voice Processing

Optimizes the wearer's natural-sounding own voice.

As a result, YourSound predicts and highlights what truly matters to Signia Xperience hearing aid wearers in any given situation.

New performance levels

7X/5X/3X



Pure 312 X



Pure Charge&Go X with the Inductive Charger II



For more details, see pages 14-16.

All your clients' needs, all together.

Signia app: greater convenience

Simpler all round, more convenient for all, the new Signia app combines all Signia's existing apps into one unified environment for all the user's needs. And because it includes TeleCare, it integrates seamlessly into your fitting process, making it easier for your customers to stay in touch with you when they are away from your premises. Suitable for use with all Signia hearing aids, its smart user interface only shows the user functions applicable to their own devices.



The app is compatible with Android and iOS. Their download on the Apple App Store or Google Play Store is free of charge.



Functionality at a glance

TeleCare



- Stay in closer contact with your customers:
- Real-time contact via text message, voice call or video call
 - Intuitive monitoring of your customers' daily satisfaction
 - Set virtual appointments for the precise remote adjustment of the hearing aids

Remote Control



- Personalize your customers' hearing experience further:
- Hearing program and volume control
 - Sound Balance Adjustment
 - 360° Spatial Configurator for individual adjustment of directionality
 - Continuous monitoring of the hearing aids' connection and battery status

Connectivity & Streaming



- Easy management of audio streaming accessories (e.g. StreamLine TV, StreamLine Mic) as well as CROS devices
- Volume level control while streaming
- Adaptive streaming volume

New performance levels for even more outstanding soundscape processing.

The sophisticated chip technology of Signia Nx packs even more performance into even less space. Its leading technology enables advanced connectivity in the smallest sized hearing aid with Ultra HD e2e and Bluetooth.

New!
Performance levels

Signia Nx is now available across all performance levels from 7 to 1 in six additional devices, so you can offer all your customers outstanding hearing performance.

New performance levels

- 7Nx
- 5Nx
- 3Nx
- New! 2Nx
- New! 1Nx



Pure 312 Nx, 13 Nx and 10 Nx hearing aids together with the Motion 13 Nx, Motion 13P Nx and Silk Nx have been extended to cover all performance ranges from performance levels 7 to 1, expanding the range of devices available with Nx technology. That means more choice for your customers, more opportunities for you.

For more details, see pages 19–26.

The ultimate rechargeable RIC.



New!
Model



Key features

- Most natural own voice and best speech understanding in noise, even when the wearer is in motion
- World's first integrated acoustic-motion sensors
- Long-lasting high quality audio streaming
- Ultra HD e2e and Bluetooth connectivity



Signia app ready.

- TeleCare
- Remote Control
- Connectivity & Streaming



Rechargeable.



Equipped with Bluetooth.



Integrated acoustic-motion sensors.

Performance levels

7

5

3

sDemo

Pure Charge&Go X.

Pure™ Charge&Go X combines all the advantages of Signia Xperience with Li-ion rechargeability and fully-featured Bluetooth connectivity. Its 20% extra battery capacity provides full flexibility and convenience for higher wearer satisfaction, with streaming of music, calls and TV in high-quality sound. All in a remarkably small design. Simply Charge&Go!

RIC

Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
				
Performance				
Matrix (2 ccm)	45/108 dB	60/119 dB	70/124 dB	75/130 dB
Battery				
Type	Lithium-ion power cell	Lithium-ion power cell	Lithium-ion power cell	Lithium-ion power cell
Power consumption	1.7 mA	1.9 mA	1.8 mA	1.8 mA



Convenience all round

The Inductive Charger II's lid protects hearing aids while they are charging. Simply pop the hearing aids in the wells and the smart charger takes care of the rest.

Smart looking, smart charging. **Inductive Charger II.**

Pure Charge&Go X comes with a new inductive charger that features a charger lid to protect the hearing aids. Its smart design also accommodates custom molds. A fast charging option, a drying function via inductive charging, and intuitive left and right LEDs for status information complete this next generation charger.

The Inductive Charger II is compatible with Pure Charge&Go X, Pure Charge&Go Nx, Motion Charge&Go Nx, Cellion primax.

The most discreet personalized hearing with direct streaming.



New!

Model



Performance levels

7

5

3

sDemo

Pure 312 X.

Equipped with all the benefits of the Signia Xperience platform, Pure 312 X attracts wearers with its elegantly slim design. Despite its discreet form, Pure 312 X provides fully-featured Bluetooth connectivity for maximum enjoyment of music, calls and TV. An integrated T-coil is also available.

Key features

- Most natural own voice and best speech understanding in noise, even when the wearer is in motion
- World's first integrated acoustic-motion sensors
- Long-lasting high quality audio streaming
- Ultra HD e2e and Bluetooth connectivity



Signia app ready.

- TeleCare
- Remote Control
- Connectivity & Streaming



Equipped with Bluetooth.



Standard housing can be exchanged for one with integrated T-Coil via Color Conversion Kits.



Integrated acoustic-motion sensors.

RIC

Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
Performance				
Matrix (2 ccm)	45/108 dB	60/119 dB	70/124 dB	75/130 dB
Battery				
Type	312	312	312	312
Power consumption	1.7 mA	1.9 mA	1.8 mA	1.8 mA

Supreme style and streaming with charging-on-the-go.



Key features

- Stylishly designed new form factor
- Most natural own voice and best speech understanding in noise thanks to Nx chip



Signia app ready.

- TeleCare
- Remote Control
- Connectivity & Streaming




Rechargeable.



Equipped with Bluetooth.



StreamLine TV
With StreamLine TV, the TV signal is directly streamed into Signia Styletto Connect.

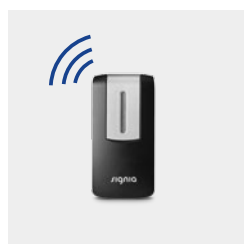

StreamLine Mic
The StreamLine Mic enables hands-free audio streaming (phone calls, music, etc.) from any device with Bluetooth connectivity.

Performance levels

7

5

3

sDemo

Signia Styletto Connect.

Signia's new Styletto Connect combines all the best of audiology, convenience and design. Powered by Signia's groundbreaking Nx platform, this fashionable hearwear combines lithium-ion power cells with all benefits of Bluetooth connectivity for recharging and streaming-on-the-go. In addition to its high-class performance, Styletto Connect comes in an exceptional and distinctive new design - with a unique appeal that will attract even more customers.

SLIM-RIC

Model	
Receiver type	miniReceiver 2.0
	
Performance	
Matrix (2 ccm)	60/119 dB
Battery	
Type	Lithium-ion power cell
Power consumption	1.4 mA

Embracing life with stylish high-tech.



Performance levels

7

5

3

2

1

sDemo



Signia Styletto.

Transforming the image of hearing aids, Signia's revolutionary hearwear Styletto combines most of the audiological advantages of our acclaimed Signia Nx platform with a stylishly designed form factor. And now it is accessible to even more people: Available in all performance levels, it provides the latest advancements in terms of energy-efficient chip design, unique lithium-ion power cells, and natural sound. All in a refined slim-line styling.

Key features

- Stylishly designed form factor
- Outstanding speech understanding in noise

Signia app ready.

- TeleCare
- Remote Control

Rechargeable.



Charging case
Wearers enjoy full-day wearing and charging-on-the-go via Signia Styletto's ultra-slim portable charging case.

SLIM-RIC	
Model	
Receiver type	miniReceiver 2.0
Performance	
Matrix (2 ccm)	60/119 dB
Battery	
Type	Lithium-ion power cell
Power consumption	1.4 mA

Convenient wireless rechargeability and Bluetooth on all levels.



Performance levels

7Nx

5Nx

3Nx

2Nx

1Nx

sDemo



Pure Charge&Go Nx.

Simply Charge&Go: Now, Signia's popular Pure Charge&Go Nx with convenient wireless lithium-ion rechargeability, is accessible for an even broader range of users. Also available in all performance levels, it provides top Bluetooth connectivity, remote support with Signia TeleCare and the most high-tech wireless charging in a small design. The proven Nx Fit fitting formula will let wearers feel its simplicity right from the start!

Key features

- Rechargeable lithium-ion power cell
- Long-lasting high quality audio streaming



Signia app ready.

- TeleCare
- Remote Control
- Connectivity & Streaming



Rechargeable.




Equipped with Bluetooth.



StreamLine TV

With StreamLine TV, the TV signal is directly streamed into Signia Styletto Connect.

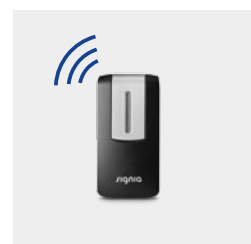

StreamLine Mic

The StreamLine Mic enables hands-free audio streaming (phone calls, music, etc.) from any device with Bluetooth connectivity.

RIC

Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
Performance				
Matrix (2 ccm)	45/108 dB	60/119 dB	70/124 dB	75/130 dB
Battery				
Type	Lithium-ion power cell	Lithium-ion power cell	Lithium-ion power cell	Lithium-ion power cell
Power consumption	1.2 mA	1.4 mA	1.3 mA	1.3 mA

Small and fully-featured with direct streaming.

New!

Performance levels

OWN VOICE PROCESSING
OVP
ULTRA HD e2e

New performance levels

7Nx

5Nx

3Nx

New! 2Nx

New! 1Nx

sDemo



Pure 312 Nx.

Equipped with all benefits of the Signia Nx platform, Pure 312 Nx keeps the wearer's own voice and remaining soundscape in perfect balance. This way, it offers the most natural own voice and best speech understanding in noise in a small, elegant housing. Despite its discreet design, Pure 312 Nx provides supreme connectivity via Bluetooth-based direct streaming and the Signia app*. Plus, TeleCare lets your customers enjoy full live remote support and fast, smooth trials.

* For Android and iOS.

Key features

- Most natural own voice and best speech understanding in noise thanks to Nx chip
- Small, discreet design
- Ultra HD e2e and Bluetooth connectivity



Signia app ready.


- TeleCare
- Remote Control
- Connectivity & Streaming



Compatible with CROS device for people with unaidable hearing loss on one side.





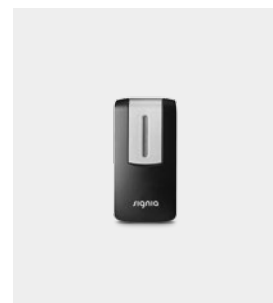

StreamLine TV
With StreamLine TV, the TV signal is directly streamed into Pure 312 Nx hearing aids.



StreamLine Mic
The StreamLine Mic enables hands-free audio streaming (phone calls, music, etc.) from any device with Bluetooth connectivity.

RIC

Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
				
Performance				
Matrix (2 ccm)	45/108 dB	60/119dB	70/124 dB	75/130 dB
Battery				
Size	312	312	312	312
Power consumption	1.2 mA	1.4 mA	1.3 mA	1.3 mA

Superior connectivity combined with the longest streaming time.

New!

Performance levels

OWN VOICE PROCESSING
OVP
ULTRA HD e2e

New performance levels

7Nx

5Nx

3Nx

New! 2Nx

New! 1Nx

sDemo



Pure 13 Nx.

By separately processing the wearer's own voice and integrating it smoothly into the soundscape, Signia's Pure 13 Nx provides the most natural own voice and best speech understanding in noise. Thanks to Bluetooth, it comes with superior connectivity with direct streaming and the Signia app*. Users enjoy the longest wearing time while streaming. Full live remote support via TeleCare quickly familiarizes them with their Pure 13 Nx hearing aids.

* For Android and iOS.

Key features

- Most natural own voice and best speech understanding in noise thanks to Nx chip
- Longest wearing time while streaming
- Ultra HD e2e and Bluetooth connectivity



Signia app ready.

- TeleCare
- Remote Control
- Connectivity & Streaming




Standard battery door can be exchanged for one with an integrated T-coil.



Compatible with CROS device for people with unaidable hearing loss on one side.



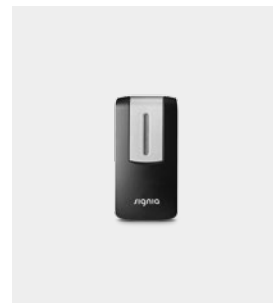


StreamLine TV
With StreamLine TV, the TV signal is directly streamed into Pure 13 Nx hearing aids.



StreamLine Mic
The StreamLine Mic enables hands-free audio streaming (phone calls, music, etc.) from any device with Bluetooth connectivity.

RIC

Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
				
Performance				
Matrix (2 ccm)	45/108 dB	60/119 dB	70/124 dB	75/130 dB
Battery				
Size	13	13	13	13
Power consumption	1.2 mA	1.4 mA	1.3 mA	1.3 mA

Simplicity with a naturally clear sound.

New!

 Performance
levels

New performance levels

7Nx

5Nx

3Nx

 New!
2Nx

 New!
1Nx

sDemo




Key features

- Ultra-small design
- Outstanding speech understanding in noise
- Remote control option



Signia app ready.

- TeleCare
- Remote Control

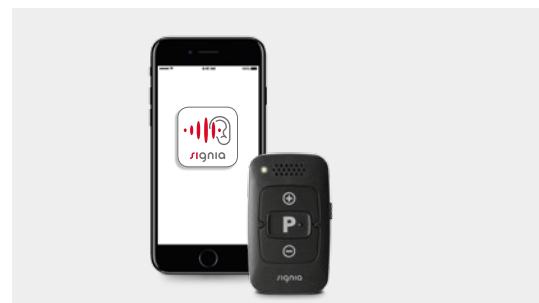

Signia app and miniPocket

To enhance its discretion, Pure 10 Nx is remotely controllable via the Signia app and the miniPocket.

Pure 10 Nx.

The ultra-small Pure 10 Nx expertly combines a highly discreet design with a more natural sound. Its sleek appearance is complemented by a wealth of audiological innovations that produce a clear sound and excellent speech understanding in noise. The non-wireless Pure 10 Nx is the ideal solution for those who wish their hearing aids to be simple and discreet.

RIC

Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
				
Performance				
Matrix (2 ccm)	45/108 dB	55/113 dB	60/118 dB	65/123 dB
Battery				
Size	10	10	10	10
Power consumption	1.2 mA	1.3 mA	1.2 mA	1.2 mA

The ultimate rechargeable BTE with Bluetooth for almost all needs.

OWN VOICE PROCESSING
OVP

ULTRA HD
e2e

Performance levels

7Nx

5Nx

3Nx

2Nx


1Nx

sDemo





Key features

- Rechargeable lithium-ion power cell
- Long-lasting high quality audio streaming

 Signia app ready.

- TeleCare
- Remote Control
- Connectivity & Streaming

 Rechargeable.

 Equipped with Bluetooth.





StreamLine TV
With StreamLine TV, the TV signal is directly streamed into Motion Charge&Go Nx hearing aids.



StreamLine Mic
The StreamLine Mic enables hands-free audio streaming (phone calls, music, etc.) from any device with Bluetooth connectivity.

Motion Charge&Go Nx.

With Motion Charge&Go Nx, Signia’s all-round BTE just got even more versatile: Now also available in all performance levels, these models combine all the advantages of Signia Nx with Li-ion powered wireless rechargeability and seamless Bluetooth connectivity in a dependable design. Intuitive to handle, their outstanding sound quality, convenient streaming options, and easy fittings make Motion Charge&Go Nx hearing aids the optimum solution for almost any customer.

BTE		
Model	Motion Charge&Go Nx	Motion Charge&Go Nx
		
Performance	ThinTube 56/126 dB	Earhook 63/134 dB
Battery	Lithium-ion power cell	Lithium-ion power cell
Type	Lithium-ion power cell	Lithium-ion power cell
Power consumption	1.4 - 2.0 mA	1.4 - 2.0 mA

Direct streaming with the most versatile fitting options.

New!

Performance levels

OWN VOICE PROCESSING
OVP

ULTRA HD
e2e

New performance levels

7Nx

5Nx

3Nx

New! 2Nx

New! 1Nx

sDemo



Motion 13 Nx.

Available in a small model and a sleek P model, the incredibly versatile Motion 13 Nx BTE is ideal for wearers with individual fitting requirements and moderate to severe or even profound hearing loss. It harmonizes the wearer's voice and surrounding sounds, providing the most natural own voice and best speech understanding in noise. Motion 13 Nx and Motion 13P Nx provide full Bluetooth connectivity and Signia TeleCare with an optional modular T-coil integrated in the battery door.

Key features

- Most natural own voice and best speech understanding in noise thanks to Nx chip
- For individual handling and fitting requirements
- Ultra HD e2e and Bluetooth connectivity



Signia app ready.

- TeleCare
- Remote Control
- Connectivity & Streaming



Standard battery door can be exchanged for one with an integrated T-coil.



Compatible with CROS device for people with unaidable hearing loss on one side.



StreamLine TV
With StreamLine TV, the TV signal is directly streamed into Motion 13 Nx hearing aids.



StreamLine Mic
The StreamLine Mic enables hands-free audio streaming (phone calls, music, etc.) from any device with Bluetooth connectivity.

BTE

Model	Motion 13 Nx	Motion 13P Nx
		
Performance	ThinTube 56/126 dB Earhook 63/134 dB	ThinTube 65/129 dB Earhook 77/135 dB
Battery		
Size	13	13
Power consumption	1.4 - 2.0 mA	1.7 - 2.0 mA

A practically invisible natural experience. Simply fit and go.

New!

Performance levels



New performance levels

7Nx

5Nx

3Nx

New! 2Nx

New! 1Nx

sDemo



Silk Nx.

The Silk™ Nx delivers a natural hearing experience in its discreet size, instant fit and exceptional sound quality. The only ready-to-wear CIC with Ultra HD e2e for superior wireless binaural processing is also 20% smaller than Silk primax™ for ultimate discretion. Thanks to its flexible silicone Click Sleeves, Silk Nx offers a precise fit without waiting times. And for wearers with unaidable hearing loss in one ear, CROS Silk Nx provides the most discreet solution – the world's first CROS CIC hearing aid.

Key features

- Best speech understanding in noise and a most natural sound thanks to Nx chip
- Click Sleeves for a quick and easy fit
- Even smaller than its predecessor
- Invisible and discreet
- Binaural OneMic directionality
- Ultra HD e2e for superior speech understanding and a most natural sound




Signia app ready.

- TeleCare
- Remote Control




Compatible with CROS device for people with unaidable hearing loss on one side.

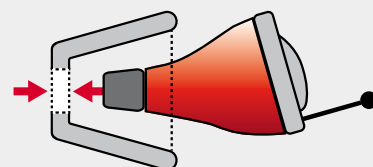


One click and you're done.

Available in four sizes, Click Sleeves simply click on to Silk Nx hearing aids and sit securely in the ear.

Click ITE

Model	
	Click CIC
	
Performance	
Matrix (2 ccm)	50/113 dB
Battery	
Size	10
Power consumption	1.3 mA

The most discreet natural hearing experience with full connectivity.

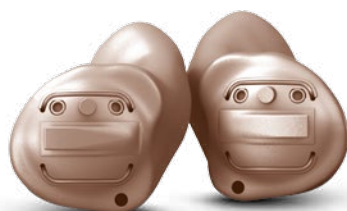

Performance levels

7Nx

5Nx

3Nx

Insio Nx.

Signia's Insio™ Nx ITC and ITE models provide top Bluetooth connectivity with best-in-class sound quality. Tailor-made for the most discerning wearers, Insio Nx delivers a more natural sound with maximum discretion. Equipped with Ultra HD e2e for superior wireless binaural processing, including as a small CIC with a single microphone, Insio offers crystal clear sound and unrivalled speech understanding even in very noisy environments.

Key features

- Best speech understanding in noise and a most natural sound thanks to Nx chip
- Small size up to high power matrix
- Full wireless and remote control options for all models
- Ultra HD e2e for superior speech understanding and a most natural sound



Signia app ready.

- TeleCare
- Remote Control
- Connectivity & Streaming*



Compatible with CROS device for people with unaidable hearing loss on one side.



Equipped with Bluetooth.*



StreamLine TV
With StreamLine TV, the TV signal is directly streamed into Bluetooth-enabled Insio Nx ITC and ITE hearing aids.



StreamLine Mic
The StreamLine Mic enables hands-free audio streaming (phone calls, music, etc.) from any Bluetooth device for Bluetooth-enabled Insio Nx ITC/ITEs.

ITE

Model	IIC	CIC	ITC	ITE
Performance				
Matrix (2 ccm)	50/113 dB ¹	S 50/113 dB ¹ M 55/118 dB ¹ P 65/124 dB ¹	S 50/113 dB ¹ M 55/118 dB ¹ P 65/124 dB ¹	M 55/118 dB ¹ P 65/124 dB ¹
Battery				
Size	10	10	312	312
Power consumption	1.3 mA ¹	1.3 mA ¹	1.3 mA ¹	1.3 mA ¹

(1) Max. values

* Available with Bluetooth-enabled Insio Nx ITC and ITE hearing aids

Discreet.
Powerful.
Effortless.

Key features

- Ultra-small design
- HD Binaural Hearing
- Clinically proven to reduce hearing effort*
- Battery size 10



Signia app ready.

- TeleCare
- Remote Control



With Notch Therapy
for tinnitus treatment.

* See legal note on final page of this document.



Performance levels

7px

5px

3px

2px

1px

Ace primax.

Signia’s Ace primax™ skillfully combines the three main elements of discretion: tiny size, automatic functionality and remote control options. Equipped with sophisticated primax star technology, it offers outstanding speech understanding and is an ideal solution for those who wish for a high level of discretion in a hearing aid.

sDemo

RIC

Model				
Receiver type				
	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
Performance				
Matrix (2 ccm)	45/108 dB	55/113 dB	60/118 dB	65/123 dB
Battery				
Size	10	10	10	10
Power consumption	0.9 mA	1.0 mA	0.9 mA	0.9 mA

Fully featured. Binaural. Effortless.



Performance levels

7px

5px

3px

2px

1px

Pure primax.

Equipped with the outstanding primax star platform, Signia's Pure primax™ provides outstanding sound comfort and sophisticated features. Its elegant, discreet housing design also allows for a quick and easy exchange of batteries. Pure primax is rechargeable via the Connexx Smart Power* charger, remotely controllable via Signia app and offers wearers full connectivity for audio streaming via Smart Connect.

Key features

- Rechargeable
- Wireless connectivity
- Battery size 312
- HD Binaural Hearing
- Clinically proven to reduce hearing effort*



Signia app ready.

- TeleCare
- Remote Control
- Audio Streaming via Smart Connect & Smart Connect app



Compatible with CROS device for people with unaidable hearing loss on one side.



Rechargeable.

* See legal note on final page of this document.







Connexx Smart Power

The Connexx Smart Power recharges and dehumidifies Pure primax hearing aids overnight, so your clients don't need to worry about handling batteries.

sDemo

* Available with performance levels 7px, 5px and 3px only.

RIC

Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
				
Performance				
Matrix (2 ccm)	45/108 dB	60/119 dB	70/124 dB	75/130 dB
Battery				
Size	312	312	312	312
Power consumption	0.9 mA	1.0 mA	1.3 mA	1.3 mA

Revolutionary battery-free hearing.



Performance levels

7px

5px

3px

2px

Cellion primax.

Signia’s groundbreaking Cellion primax™ offers revolutionary lithium-ion technology in a unique, contact-free OneShell design with no battery door. Once fully charged via the inductive charger, the high-capacity power cell makes Cellion primax the only hearing aids that last all day while offering 24-hour continuous streaming via Smart Connect.

Key features

- Rechargeable lithium-ion power cell
- 24 hours continuous use with unlimited streaming
- Contactless, inductive charging
- HD Binaural Hearing
- Clinically proven to reduce hearing effort*



Signia app ready.

- TeleCare
- Remote Control
- Audio Streaming via Smart Connect & Smart Connect app



Rechargeable.



Equipped with an integrated T-coil.



With Notch Therapy for tinnitus treatment.





* See legal note on final page of this document.



Inductive Charger II

The Inductive Charger II and dehumidifier with its fast charging option is an ideal companion to Cellion primax.

sDemo

RIC				
Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
				
Performance				
Matrix (2 ccm)	45/108 dB	60/118 dB	70/124 dB	75/130 dB
Battery				
Type	Lithium-ion power cell	Lithium-ion power cell	Lithium-ion power cell	Lithium-ion power cell
Power consumption	0.9 mA	1.0 mA	1.0 mA	1.1 mA

Power meets versatility.



Key features

- Rechargeable
- Battery size 13
- Wireless connectivity
- HD Binaural Hearing
- Clinically proven to reduce hearing effort*



Signia app ready.

- TeleCare
- Remote Control
- Audio Streaming via Smart Connect & Smart Connect app



Rechargeable.



With Notch Therapy for tinnitus treatment.

* See legal note on final page of this document.

Performance levels

7px

5px




3px

Carat primax.

Signia's Carat primax™ was designed to deliver optimum amplification for almost all levels of hearing loss. Its optional rechargeable battery packs longer lasting power, so wearers can enjoy highly convenient handling. Carat primax' remote control and wireless streaming options make these hearing aids a comfortable and discreet solution.

sDemo

RIC

Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
				
Performance				
Matrix (2 ccm)	45/108 dB	60/119 dB	70/124 dB	75/130 dB
Battery				
Size	13	13	13	13
Power consumption	0.9 mA	1.1 mA	1.0 mA	1.1 mA

Sleek. Convenient. Effortless.



Key features

- Sleek and compact housing
- Easy handling
- HD Binaural Hearing
- Clinically proven to reduce hearing effort*
- Option for Motion SA, P and SP primax models: Direct Audio Input via audio shoe



Signia app ready.

- TeleCare
- Remote Control
- Audio Streaming via Smart Connect & Smart Connect app



Rechargeable.



With Notch Therapy for tinnitus treatment.



Direct Audio Input.

* See legal note on final page of this document.

Performance levels

7px

5px

3px

2px





1px

sDemo

Motion primax.

Motion primax™ from Signia comes in a broad and versatile BTE range. From the small S model to the sleek but powerful P model, from the SA model that can be equipped with an audio shoe to the rechargeable SX to the super power SP model, Motion primax provides a solution for almost every ear and every hearing loss. For a maximum level of convenience, all models of the Motion family are remotely controllable and offer full connectivity for audio streaming via Smart Connect.

BTE

Model	Motion S primax	Motion SX/SA primax	Motion P primax	Motion SP primax
				
Performance	LifeTube 45/124 dB Earhook 55/124 dB	ThinTube 53/126 dB Earhook 60/130 dB	ThinTube 65/129 dB Earhook 77/135 dB	Earhook 82/140 dB
Battery				
Size	312	13	13	675
Power consumption	0.8 mA	1.0 mA	1.1 - 1.3 mA	2.1 mA

Click.
Fit.
Go.

Performance levels

- 7px
- 5px
- 3px
- 2px
- 1px

sDemo



Silk primax.

Delivering the full convenience of Click CICs, the nearly invisible Signia Silk primax lets you provide the smoothest, fastest ITE fittings. Thanks to the flexible, silicone Click Sleeves, Silk primax Click models offer a precise fit, binaural OneMic directionality and full wireless connectivity. This makes them ideal for first-time wearers looking for a discreet on-the-spot solution without any fuss.

Key features

- Click Sleeves for a swift fit
- 3 sleeve sizes to fit every ear
- Invisible and discreet
- Binaural OneMic directionality
- Clinically proven to reduce hearing effort*




Signia app ready.

- TeleCare
- Remote Control
- Audio Streaming via Smart Connect & Smart Connect app




With Notch Therapy for tinnitus treatment.

* See legal note on final page of this document.

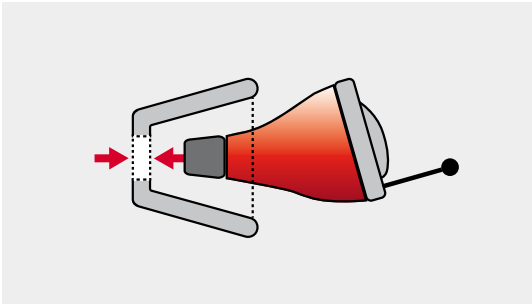

One click and you're done.
Click Sleeves simply click on to Silk primax hearing aids and sit securely in the ear.

Click ITE	
Model	Click CIC
	
Performance	
Matrix (2 ccm)	50/113 dB
Battery	
Size	10
Power consumption	1.1 mA

Ultra-discreet. Custom fit. Effortless.

Key features

- Binaural OneMic directionality
- Small size up to high power matrix
- Full wireless and remote control options for all models
- Power matrix in all models, from IICs to ITEs
- Clinically proven to reduce hearing effort*




Signia app ready.

- TeleCare
- Remote Control
- Audio Streaming via Smart Connect & Smart Connect app




With Notch Therapy for tinnitus treatment.

* See legal note on final page of this document.

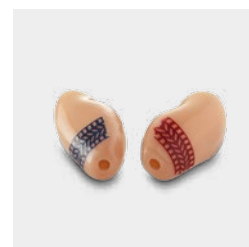

CanalGrip®
The CanalGrip® is a durable pattern for retention in difficult ear canals.

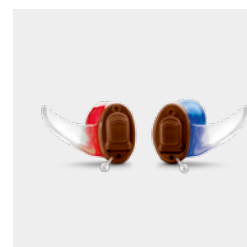

Concha Lock
The Concha Lock provides a precise and stable retention. The transparent color supports high discretion.

Performance levels

7px

5px

3px

2px

1px

Insio primax.

Tailor-made for demanding wearers, Insio primax™ from Signia offers the ultimate in discretion and performance. Thanks to their binaural OneMic directionality function, even the small CIC models with a single microphone deliver superior directionality to the front. All models can be controlled remotely for a maximum of discretion.

ITE

Model	IIC	CIC	ITC	ITE
Performance				
Matrix (2 ccm)	65/124 dB ¹	65/124 dB ¹	65/124 dB ¹	65/124 dB ¹
Battery				
Size	10	10	312/10	13/312
Power consumption	1.1 mA ¹	1.1 mA ¹	1.1 mA ¹	1.1 mA ¹

(1) Max. values

Small and powerful: the first Intuis 3 RIC.

Key features

- For mild to severe hearing loss
- Small design for a high level of wearing comfort



Signia app ready.

- TeleCare
- Remote Control



Signia app and miniPocket

To enhance its discretion, Intuis 3 RIC312 is remotely controllable via the Signia app™ and the miniPocket™.





Performance level

Essential

Intuis 3 RIC312.

Welcome to the family! Completing the proven Signia Intuis 3 range, the remotely controllable Intuis 3 RIC312 is an attractive and convenient solution for mild to severe hearing loss. Thanks to its automatic directionality and a variety of noise reduction features, it offers a reliable hearing experience and improved speech intelligibility even in noisy environments. TeleCare lets your customers enjoy live remote support and fast, smooth trials.

RIC

Model				
Receiver type	miniReceiver 2.0 S	miniReceiver 2.0 M	miniReceiver 2.0 P	miniReceiver 2.0 HP
				
Performance				
Matrix (2 ccm)	45/108 dB	60/119 dB	70/124 dB	75/130 dB
Battery				
Type	312	312	312	312
Power consumption	0.9 mA	1.0 mA	1.0 mA	1.1 mA

Proven technology, outstanding comfort.

Key features

- For all levels of hearing loss
- Ergonomic design for a high level of wearing comfort
- Remotely controllable for a maximum of convenience and discretion



Signia app ready.





- TeleCare
- Remote Control



Performance level
Essential

Intuis 3 BTEs.

All Signia’s Intuis 3 BTE models offer proven technology with 6 channel MPO for the most advanced fitting options and for more individual adjustments. Suitable for all levels of hearing loss, the easy to handle Intuis 3 BTEs are equipped with automatic directional microphones for a precise hearing experience, offer remote control options, and are TeleCare ready for smoother, faster trials and higher customer satisfaction.

BTE				
Model	Intuis 3 S	Intuis 3 M	Intuis 3 P	Intuis 3 SP
Performance				
Matrix (2 ccm)	LifeTube 45/124 dB Earhook 55/124 dB	LifeTube 53/125 dB Earhook 60/130 dB	Earhook 70/134 dB	Earhook 80/140 dB
Battery	312	13	13	675
Size	0.9 mA	1.0 mA	1.4 mA	2.4 mA
Power consumption				

Key features

- For mild to severe hearing loss
- 6 channel MPO for the most advanced fitting options and more individual adjustments
- Remotely controllable for a maximum of discretion and convenience



Signia app ready.



- TeleCare
- Remote Control



Performance level
Essential

Intuis 3 Click ITEs and ITEs.

With Intuis 3 Click ITEs and tailor-made ITEs, you can offer your customers the full range of in-the-ear models: No matter if they wish for a solution that fits right away or prefer a customized model, Intuis 3 is the perfect match. Equipped with proven technology, all models offer a precise hearing experience, are remotely controllable for a maximum of discretion and TeleCare ready for successful trials.

Click ITE/ITE		
Model	<div>Intuis 3 Click CIC, Click ITC</div> <div></div>	
Performance	<div>Intuis 3 CIC, ITC, ITE</div> <div></div>	
Matrix (2 ccm)	Click CIC 50/113 dB Click ITC 55/116 dB	CIC 65/124 dB ¹ ITC 65/124 dB ¹ ITE 65/124 dB ¹
Battery		
Size	Click CIC 10, Click ITC 312	CIC 10, ITC 312, ITE 13
Power consumption	1.1 mA	1.1 mA ¹

(1) Max. values

Make your customers the center of attention.

Key features

- A full range of BTE, Click ITE and ITE models
- For all levels of hearing loss
- Remotely controllable for a maximum of convenience and discretion



Signia app ready.

- TeleCare
- Remote Control

Lotus

The Prompt family.

Signia's Prompt family offers outstanding wearing comfort and ease of use for every form of hearing loss. All models offer remote control options via the Signia app and the miniPocket™ and are a convenient solution for almost all your customers.



BTE

Model	Prompt S	Prompt P	Prompt SP
Performance			
Matrix (2 ccm)	ThinTube 45/124 dB Earhook 55/124 dB	Earhook 70/134 dB	Earhook 80/140 dB
Battery			
Size	312	13	675
Power consumption	0.9 mA	1.4 mA	2.4 mA

Click ITE/ITE

Model	Prompt Click CIC, Click ITC	Prompt CIC, ITC, ITE
Performance		
Matrix (2 ccm)	Click CIC 50/113 dB Click ITC 55/116 dB	CIC 65/124 dB ¹ ITC 65/124 dB ¹ ITE 65/124 dB ¹
Battery		
Size	Click CIC 10, Click ITC 312	CIC 10, ITC 312/10, ITE 13/312
Power consumption	1.1 mA	1.1 mA ¹

(1) Max. values

Take hearing to a new level.

Key features

- Quick and easy fittings everywhere via the MobileFitting app
- For mild to profound hearing loss
- Ergonomic shape for a high level of wearing comfort



Signia app ready.

- TeleCare
- Remote Control




Lotus

Run.



Easy to fit in just a few minutes with the convenient MobileFitting app, Run from Signia is ideal for on-the-spot fittings. Incredibly comfortable thanks to its ergonomic shape, the convenient and easy-to-use Run delivers an outstanding hearing experience.



BTE

Model	Run S	Run P	Run SP
			
Performance	ThinTube 45/124 dB Earhook 55/124 dB	Earhook 70/132 dB	Earhook 79/140 dB
Battery			
Size	312	13	675
Power consumption	0.9 mA	1.3 mA	2.2 mA

Click ITE

Model	Run Click CIC	Run Click ITC
		
Performance		
Matrix (2 ccm)	50/113 dB	55/116 dB
Battery		
Size	10	312
Power consumption	1.1 mA	1.1 mA

Provide an outstanding hearing experience.

- Key features
- A high level of amplification for severe to profound hearing loss
 - Ergonomic shape for a high level of wearing comfort
 - Simple fittings everywhere via the rocker switch

Lotus

Fun.

Equipped with Signia’s proven hearing aid technology, Fun covers all essential customer requirements. It offers superb sound quality for severe to profound hearing loss, excellent wearing comfort and is simply fitted by pushing the handy rocker switch.



BTE		
Model		
	<div>Fun P</div> 	<div>Fun SP</div> 
Performance		
Matrix (2 ccm)	Earhook 70/132 dB	Earhook 79/140 dB
Battery		
Size	13	675
Power consumption	1.2 mA	2.2 mA

Deliver a powerful listening experience.

Key features


- Simple fittings everywhere via the rocker switch
- For mild to severe hearing loss
- Easy handling

Lotus

Fast.

Fast BTE hearing aids offer a high level of performance and deliver just the right amount of amplification for customers with severe hearing loss. They are easily adjusted via the rocker switch for simplified fitting. Sleek and comfortable with a range of features, Signia's Fast ensures an outstanding hearing experience suitable for every customer.



BTE	
Model	Fast P
	
Performance	
Matrix (2 ccm)	Earhook 62/130 dB
Battery	
Size	13
Power consumption	1.1 mA


Sharpening your customers' senses.

SecureEar.

SecureEar™ is an active digital hearing protection that prevents the wearer from excessive noise and loud sounds. Customized for a perfect fit, it ideally combines hearing protection with the ability to communicate effectively in all environments. With SecureEar, the wearer can also receive cell phone or CB calls at any time thanks to the integrated T-coil.



SecureEar with T-coil: Protects hearing and also allows the user to communicate without any constraints.

ITE	
Model	SecureEar
	
Performance	
Matrix (2 ccm)	85 dB
Battery	
Size	312
Battery life	~ 230 h


Signia app overview.



One smartphone app for all devices. The new Signia app gives users all the convenience and discretion they need in one unified app. One download is all they need to control their hearing aids, stream music from their phones, TVs and other compatible devices or even get in touch with you via the integrated TeleCare functions.



All apps are compatible with Android and iOS. Their download on the Apple App Store or Google Play Store is free of charge.

	 Signia app
Availability	Apple App Store/ Google Play Store
Use case & compatibility	All Signia hearing aids (except Fun and Fast)
Remote control features	
Transmission signal	Bluetooth + High frequency coded signal
Volume adjustment	●
Independent volume adjustment (R/L)	● ^A
Program change	●
SoundBalance adjustment	●
Adjustable system sounds	● ^{B,D}
Spatial configurator adjustment	● ^C
Power-on-Delay configuration	● ^B
Battery status read-out	●
Low battery interval	● ^D
Adaptive streaming volume	● ^E
Motion sensor usage	● ^{B,F}
Signia TeleCare features ^G	
Daily satisfaction rating	●
Hearing lessons	● ^D
CareChat - Text message	●
CareChat - Phone call	● ^{H,I}
CareChat - Video call	● ^{H,J}
Private label option	● ^{D,K}

- A) Configurable via Connexx

B) Available with Bluetooth-enabled hearing aids, only

C) Available with hearing aids with two microphones each and at performance levels 7 and 5

D) Not available in the first app version. Will be available via an update in November 2019.

E) This functionality is automatically available (no possibility to adjust via the app)

F) Built-in motion sensors for Signia Xperience devices, only. For other Bluetooth-enabled hearing aids, the motion sensor of the smartphone will be used

G) TeleCare functionalities are available upon the activation by the Hearing Care Professional
- H) The possibility of making direct phone and video calls is available via Connexx

I) The clients can make a phone call via their regular mobile network if the Hearing Care Professional has already entered his/her phone number in the TeleCare Portal

J) Video call can be initiated by the Hearing Care Professional, only

K) Activation via customer service

Signia app feature overview.

Available for Android and iOS smartphones, the Signia app puts remote support, audio streaming and remote control at your clients' fingertips and is compatible with most Signia hearing aids. Here is an overview of the main features.

TeleCare



TeleCare is fully integrated into the Signia app for a close and personal fitting experience. You can use it for real-time video chats when your customers need your assistance. You'll get information about their current hearing situation for precise remote adjustment of their hearing aids.

Remote Control



Complete discretion and convenience guaranteed. Users have complete control over settings such as volume, bass and treble, and can change programs and optimize the settings of their hearing aids. All from their smartphone.








Connectivity & Streaming



Thanks to the Signia app, your customers can manage their streaming devices and settings in a very easy and intuitive way while listening to their favorite music and TV programs as well as receiving or making phone calls.

Apps, accessories and hearing aid compatibility.

This overview shows you which apps and accessories work with our hearing aids. Depending on the solution your customers choose, they can even enhance their hearing experience and convenience with this well-matching portfolio.

Hearing aid	Signia app	CROS/BiCROS	Streamers and transmitters		Remote controls	Charging stations	Inductive Charger II
		CROS	StreamLine TV	StreamLine Mic	miniPocket	Charger	Charger
							
Pure Charge&Go X New!	●	-	●	●	●	-	●
Pure 312 X New!	●	-	●	●	●	-	-
Styletto Connect	●	-	●	●	●	A	-
Styletto	●	-	-	-	●	B	-
Pure 10 Nx New!	●	-	-	-	●	-	-
Pure 312 Nx New!	●	C	●	●	●	-	-
Pure 13 Nx New!	●	C	●	●	●	-	-
Pure Charge&Go Nx	●	C	●	●	●	-	D
Motion 13 Nx New!	●	C	●	●	●	-	-
Motion 13P Nx New!	●	C	●	●	●	-	-
Motion Charge&Go Nx	●	C	●	●	●	-	D
Silk Nx New!	●	E	-	-	●	-	-
Insio Nx	●	G,H	F	F	●	-	-
Ace primax	●	-	-	-	●	-	-
Pure primax	●	I,J	-	-	●	-	I,K
Cellion primax	●	J,L	-	-	●	-	D
Carat primax	●	J	-	-	●	-	-
Motion SP primax	●	I,J	-	-	●	-	-
Motion P primax	●	I,J	-	-	●	-	-
Motion SX primax	●	I,J	-	-	●	-	I,K
Motion SA primax	●	I,J	-	-	●	-	-
Motion S primax	●	-	-	-	●	-	-
Silk primax	●	I,J	-	-	●	-	-
Insio primax	●	I,J	-	-	●	-	-
Intuis 3	●	-	-	-	●	-	-
Intuis 3 RIC312	●	-	-	-	●	-	-
Prompt	●	-	-	-	●	-	-
Run	●	-	-	-	●	-	-

A) Compatible with Styletto Connect charging case

B) Compatible with Styletto charging case

C) Compatible with CROS Pure 312 Nx or CROS Pure Charge&Go Nx

D) Compatible with the Pure Charge&Go/ Motion Charge&Go/Cellion charger

E) Compatible with CROS Silk Nx

F) Compatible with Bluetooth-enabled Insio Nx ITC/ITE

G) Insio Nx IIC and CIC are compatible with CROS Silk Nx

H) Bluetooth-enabled Insio Nx ITC and ITE are compatible with CROS Pure 312 Nx

I) Not available with 2px/1px

J) Compatible with CROS Pure

K) Compatible with Connexx Smart Power and Smart Charger

L) Not available with 2px

Accessory overview.

Streamers



Via **StreamLine TV**, TV sound can be fed directly into Bluetooth-equipped Signia hearing aids. Wearers can even adjust the TV volume individually via the Signia app.



The **StreamLine Mic** transforms Signia devices with Bluetooth into more than state-of-the-art hearing aids. They also become hands-free mobile headsets and wireless stereo headphones that stream highest quality audio into both ears from any Bluetooth device. Wearers can even adjust the streaming volume individually via the Signia app.

Remote controls



Small enough to carry on a keyring, the **miniPocket** is the ideal solution for anyone who wants discreet, convenient hearing aid control without the need for a large remote control or smartphone.

CROS/BiCROS



For people with unaidable hearing loss on one side, **Signia CROS** devices receive sound on that side and transmit it to a hearing aid on the other side.

Charging stations



The **chargers** recharge hearing aids so your clients don't need to worry about handling batteries. Compatible with Pure Charge&Go X, Pure Charge&Go Nx, Motion Charge&Go Nx and Cellion primax hearing aids, **Inductive Charger II** charges and dries hearing aids to give a full day's ultra-durable hearing performance.







Color overview SLIM-RIC, RICs and BTEs.

Products	Colors											
	Beige	Granite	Grey	Brown	Black	Silver	Pearl White	Golden Blonde	Sandy Brown	Dark Champagne	Candy Pink	Galactic Blue
	1	2	3	4	5	6	7	30	31	37	60	61
Pure Charge&Go X	●	●	●		●		●		●	●		
Pure 312 X	●	●	●		●		●		●	●		
Styletto Connect												
Styletto					●							
Pure 10 Nx	●	●	●		●	●	●	●	●	●		
Pure 312 Nx	●	●	●		●	●	●	●	●	●		
Pure 13 Nx	●	●	●		●	●	●	●	●	●		
Pure Charge&Go Nx	●	●	●		●	●	●	●	●	●		
Motion 13 Nx	●	●	●		●	●	●	●	●	●		
Motion 13P Nx	●	●	●		●	●	●	●	●	●		
Motion Charge&Go Nx	●	●	●		●	●	●	●	●	●		
Ace primax	●	●	●		●	●	●	●	●			
Pure primax	●	●	●		●	●	●	●	●	●		
Cellion primax	●	●	●		●	●	●	●	●	●		
Carat primax	●	●	●		●	●	●	●	●	●		
Motion primax	●	● ⁴	●		●	● ⁵		● ⁶	● ⁶	● ⁶	● ^{2,7}	● ^{2,7}
Intuis 3	●		●	●	●	●						
Intuis 3 RIC 312	●		●	●	●	●						
Prompt	●		●									
Run	●		●									
Fun	●		●									
Fast	●											

(2) Not available with 2px/1px (5) Not for Motion SP primax (7) Only for Motion SA and P primax
 (3) Not available with 2px (6) Only for Motion SX, SA (8) Only for Motion SX and SA primax
 (4) Not for Motion S primax and P primax

Color overview Click ITEs.

Standard colors for Click ITEs

Description	Type	Silk Nx	Silk primax	Intuis 3	Prompt	Run
 Blue/Mocha	left Click CIC	●	●			
 Red/Mocha	right Click CIC	●	●			
 Blue/Black	left Click CIC	●				
 Red/Black	right Click CIC	●				
 Mocha/Beige	Click CIC			●	●	●
 Tan/Beige	Click ITC			●	●	●

Further colors available on request.

Dark Granite	Deep Brown	Deep Red	White	Silver (new)	Rose Gold	Spirit	Elegance	Dark Champagne/Black	Sandy Brown/Black	Black/Silver	Dark Granite/Silver	Cosmic Blue/Rose Gold	Snow White/Rose Gold
66	72	73		926	927	C5	C12	HC2	HC4				
	●			●	●								
	●			●	●								
										●		●	●
			●								●	●	●
●	●							●					
●	●								●				
●	●								●				
●	●							●					
●		●				●							
●	●	● ²				● ²							
●	●	● ³											
●	●	●				●							
●	●	● ^{2,4,5}		● ^{2,7}	● ^{2,7}	● ^{2,8}	● ^{2,8}						
●													
	●												
	●												
	●												



Color Flipper
Presenting RIC, BTE and ITE colors, and product-specific overview of available colors per product.
Item No. 03287-99C1-7600

Color overview ITEs.

Standard colors for custom-made ITEs

Description	Type	Insio Nx	Insio Nx ITC/ITE	Insio primax	Intuis 3	Prompt
Beige	ITC, ITE		●	●	●	●
Tan	ITC, ITE		●	●	●	●
Mocha	CIC, ITC, ITE	●	●	●	●	●
Brown	CIC, ITC, ITE	●	●	●	●	●
Dark Brown	IIC, ITC, ITE	●	●	●	●	●
Black	IIC, CIC	●				
Deep Red	IIC, ITC, ITE			●*		

Further colors available on request.
* Insio primax 7px/5px/3px only.



Meeting all your needs:

The Signia hearing care professional support.

Every day you try to meet and exceed your customers' hearing needs and wishes. In Signia you have a partner who fully supports you. Our broad range of hearing aid components, counseling and trial support as well as fitting tools enhances your workflow and provides you with all you need to find the best individual solution for your customers.

For detailed information, see our website signia-pro.com

Perfection is in the detail:

Components for hearing aids.



miniReceivers



Click Sleeves 2.0



Click Domes
and LifeTubes



Click Molds

From RIC miniReceivers™ to BTE earpieces and coupling options to Click Sleeves 2.0 for Click ITEs and RICs, we supply you with all necessary items so you can tailor the perfect hearing solution for your customers.

Hearing aid components

RICs*: miniReceivers S, M, P, HP
Click Sleeves 2.0 XS, S, M, L

BTEs: Click Domes™
Click Molds™
ThinTubes
LifeTubes
S-LifeTubes
LifeTips
Lotus Coupling
Earmolds

Click ITEs: Click Sleeves 2.0 XS, S, M, L

* Styletto is only compatible to M-size receivers and all sizes of Click Sleeves.

Reducing effort, increasing satisfaction:

Support for counseling, fittings and trials.



Connex 9
fitting software



Unity 3
mobile fitting unit system



SoundDemo Toolkit
incl. sDemo hearing aids









Counseling Suite
advisory tool

Signia offers you a broad range of counseling and fitting support – from programming software and tools to mobile fitting units to hearing aids for demonstration and advisory tools – up to comprehensive remote hearing care. This enhances your workflow and makes trials and fittings smoother and more pleasant for your customers.

Tools and support for counseling, fittings and trials








Connex 9
iScan™ II
Unity™ 3
ConnexAir
Noahlink
Noahlink Wireless
HI-PRO 2 USB
TeleCare
SoundDemo Toolkit

Detailed feature overview **SLIM-RIC** and **RICs**.

Type	Pure Charge&Go X		Pure 312 X	Styletto Connect	Styletto	Pure Charge&Go Nx	Pure 312 Nx
Performance levels	7X/5X/3X		7/5/3	7/5/3/2/1	7Nx/5Nx/3Nx/2Nx/1Nx		
Picture	<div>New!</div> 	<div>New!</div> 	<div>Powered by Signia Nx</div> 	<div>Powered by Signia Nx</div> 	<div>New!</div> 	<div>New!</div> 	
Product details							
Matrix (2 ccm)	S 45/108 dB M 60/119 dB P 70/124 dB HP 75/130 dB	S 45/108 dB M 60/119 dB P 70/124 dB HP 75/130 dB	- 60/119 dB - -	- 60/119 dB - -	S 45/108 dB M 60/119 dB P 70/124 dB HP 75/130 dB	S 45/108 dB M 60/119 dB P 70/124 dB HP 75/130 dB	
Battery size	Lithium-ion power cell	312	Lithium-ion power cell	Lithium-ion power cell	Lithium-ion power cell	312	
Power consumption	1.7 - 1.9 mA	1.7 - 1.9 mA	1.4 mA	1.4 mA	1.2 - 1.4 mA	1.2 - 1.4 mA	
Features							
Frequency channels/ Compression controls	7X - 48/20 5X - 32/16 3X - 24/12 - -	7X - 48/20 5X - 32/16 3X - 24/12 - -	7 - 48/20 5 - 32/16 3 - 24/12 - -	7 - 48/20 5 - 32/16 3 - 24/12 2 - 16/8 1 - 16/8	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 2Nx - 16/8 1Nx - 16/8	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 2Nx - 16/8 1Nx - 16/8	
Hearing programs	6	6	6	6	6	6	
e2e wireless	● ¹³	● ¹³	● ¹³	● ¹³	● ¹³	● ¹³	
Direct streaming	●	●	●	-	●	●	
TwinPhone	● ¹⁶	● ¹⁶	● ¹⁵	● ¹⁵	● ¹⁷	● ¹⁶	
TeleCare	● ¹⁹	● ¹⁹	● ¹⁹	● ¹⁸	● ¹⁹	● ¹⁹	
Tinnitus therapies (available on 7/5/3 PL)	●	●	●	●	●	●	
CROS/BiCROS	-	-	-	-	● ^{2, 21}	● ^{2, 21}	
Push button	-	-	-	-	●	-	
Volume control	● ²⁴	● ²⁴	-	-	● ²⁴	● ²⁴	
Rechargeable	●	-	●	●	●	-	
T-coil	-	○	-	-	-	-	
Battery door on/off function	-	●	-	-	-	●	
Audio input	● ²⁸	● ²⁸	● ²⁸	-	● ²⁸	● ²⁸	
Modifications							
Color conversion kit	○	○	-	-	○	○	
App/Accessory compatibility							
Signia app	●	●	●	●	●	●	
Smart Connect	-	-	-	-	-	-	
StreamLine TV/StreamLine Mic	●	●	●	-	●	●	
miniPocket	●	●	●	●	●	●	
Connexx Smart Remote	-	-	-	-	-	-	
ConnexxAir	-	-	-	●	-	-	
Noahlink Wireless	●	●	●	-	●	●	
Programming adapter/cable	-	-	-	-	-	size 312	
Fitting set	miniReceiver 2.0	miniReceiver 2.0	-	-	miniReceiver 2.0	miniReceiver 2.0	

- (2) Not available with PL 2/1
- (3) Not available with 2px
- (10) 6 hearing programs for 7px/5px/3px
- (11) 4 hearing programs for 2px/1px
- (12) 4 hearing programs for 2px
- (13) Ultra HD e2e
with binaural audio signal processing
- (14) HD Binaural Hearing
with binaural audio signal processing
- (15) To be activated via Signia app
or miniPocket






- (16) To be activated via rocker switch, Signia app or
miniPocket
- (17) To be activated via push button, Signia app or
miniPocket
- (18) Basic remote tuning via 4-band-equalizer
- (19) Full live remote tuning via Connexx
- (20) Not for 1px
- (21) With either CROS Pure 312 Nx or
CROS Pure Charge&Go Nx
- (22) With CROS Pure

Pure 13 Nx	Pure 10 Nx	Ace primax	Pure primax	Cellion primax	Carat primax	Intuis 3 RIC312
7Nx/5Nx/3Nx/2Nx/1Nx		7px/5px/3px/2px/1px		7px/5px/3px/2px	7px/5px/3px	Essential
<div><div>New!</div></div> <div><div>New!</div></div>						
S 45/108 dB M 60/119 dB P 70/124 dB HP 75/130 dB	S 45/108 dB M 55/113 dB P 60/118 dB HP 65/123 dB	S 45/108 dB M 55/113 dB P 60/118 dB HP 65/123 dB	S 45/108 dB M 60/119 dB P 70/124 dB HP 75/130 dB	S 45/108 dB M 60/118 dB P 70/124 dB HP 75/130 dB	S 45/108 dB M 60/119 dB P 70/124 dB HP 75/130 dB	S 45/108 dB M 60/119 dB P 70/124 dB HP 75/130 dB
13	10	10	312	Lithium-ion power cell	13	312
1.2 - 1.4 mA	1.2 - 1.3 mA	0.9 - 1.0 mA	0.9 - 1.3 mA	0.9 - 1.1 mA	0.9 - 1.1 mA	0.9 - 1.1 mA
7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 2Nx - 16/8 1Nx - 16/8	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 2Nx - 16/8 1Nx - 16/8	7px - 48/20 5px - 32/16 3px - 24/12 2px - 16/8 1px - 16/8	7px - 48/20 5px - 32/16 3px - 24/12 2px - 16/8 1px - 16/8	7px - 48/20 5px - 32/16 3px - 24/12 2px - 16/8 -	7px - 48/20 5px - 32/16 3px - 24/12 -	12/6
6	6	6 ^{10/4 11}	6 ^{10/4 11}	6 ^{10/4 12}	6	4
● ¹³	-	-	● ¹⁴	● ¹⁴	● ¹⁴	-
●	-	-	-	-	-	-
● ¹⁶	-	-	● ²	● ³	●	-
● ¹⁹	● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸
●	●	● ²⁰	● ²⁰	●	●	-
● ^{2, 21}	-	-	● ^{2, 22}	● ^{3, 22}	● ²²	-
-	○	●	● ²³	●	● ²⁴	● ²³
● ²⁴	-	● ²⁵	● ²³	● ²⁵	● ²⁴	● ²³
-	-	-	● ^{2, 26}	●	● ²⁷	-
○	-	-	●	●	●	●
●	●	●	●	-	●	●
● ²⁸	-	-	○ ²⁹	○ ²⁹	○ ²⁹	-
○	○	○	○	○	○	○
●	●	●	●	●	●	●
-	-	-	●	●	●	-
●	-	-	-	-	-	-
●	●	●	●	●	●	●
-	-	-	●	●	●	-
-	-	-	●	●	●	-
●	-	-	-	-	-	-
size 13	size 10	size 10	size 312	CS 44 cable	size 13	size 312
miniReceiver 2.0	miniReceiver 2.0	miniReceiver 2.0	miniReceiver 2.0	miniReceiver 2.0	miniReceiver 2.0	miniReceiver 2.0

- (23) Exchangeable controls - programmable via Connexx
- (24) Programmable via Connexx
- (25) With Signia app/miniPocket
- (26) Only recommended with miniReceiver S
- (27) Reduced battery life
- (28) Via direct streaming
- (29) With Smart Connect

- Standard
- Optional

Detailed feature overview **BTEs.**

Type	Motion Charge&Go Nx	Motion 13 Nx	Motion 13P Nx	Motion SP primax	Motion P primax
Performance levels	7Nx/5Nx/3Nx/2Nx/1Nx			7px/5px/3px/2px/1px	
Picture		 New!	 New!		
Product details					
Matrix (2 ccm) ThinTube/ LifeTube Earhook	56/126 dB - 63/134 dB	56/126 dB - 63/134 dB	65/129 dB - 77/135 dB	- 82/140 dB	65/129 dB - 77/135 dB
Battery size	Lithium-ion power cell			675	13
Power consumption	1.4 - 2.0 mA	1.4 - 2.0 mA	1.7 - 2.0 mA	2.1 mA	1.1 - 1.3 mA
Features					
Frequency channels/ Compression controls	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 2Nx - 16/8 1Nx - 16/8	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 2Nx - 16/8 1Nx - 16/8	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 2Nx - 16/8 1Nx - 16/8	7px - 24/16 5px - 24/16 3px - 24/12 2px - 16/8 1px - 16/8	7px - 48/20 5px - 32/16 3px - 24/12 2px - 16/8 1px - 16/8
Hearing programs	6	6	6	6 ^{10/4/11}	6 ^{10/4/11}
e2e wireless	● ¹³	● ¹³	● ¹³	● ¹⁴	● ¹⁴
Direct streaming	●	●	●	-	-
TwinPhone	● ¹⁷	● ¹⁶	● ¹⁶	● ^{2, 17}	● ^{2, 16}
TeleCare	● ¹⁹	● ¹⁹	● ¹⁹	● ¹⁸	● ¹⁸
Tinnitus therapies	●	●	●	● ²⁰	● ²⁰
CROS/BiCROS	● ^{2, 21}	● ^{2, 21}	● ^{2, 21}	● ^{2, 22}	● ^{2, 22}
Push button	●	-	-	●	-
Volume control	● ²⁴	● ²⁴	● ²⁴	● ²⁴	● ²⁴
Rechargeable	●	-	-	-	-
T-coil	-	○	○	●	●
Battery door on/off function	-	●	●	●	●
Audio input	● ²⁸	● ²⁸	● ²⁸	○ ³²	○ ³²
Modifications					
Color conversion kit	○	○	○	-	○
App/Accessory compatibility					
Signia app	●	●	●	●	●
Smart Connect	-	-	-	●	●
StreamLine TV/StreamLine Mic	●	●	●	-	-
miniPocket	●	●	●	●	●
easyPocket	-	-	-	●	●
ConnexxAir	-	-	-	●	●
Noahlink Wireless	●	●	●	-	-
Programming adapter/cable	-	size 13	size 13	CS 44 cable	size 13
Fitting set	ThinTube/Earhook	ThinTube/Earhook	ThinTube/Earhook	-	ThinTube/Earhook

- (2) Not available with PL 2/1
 (10) 6 hearing programs for 7px/5px/3px
 (11) 4 hearing programs for 2px/1px
 (13) Ultra HD e2e
 with binaural audio signal processing
 (14) HD Binaural Hearing
 with binaural audio signal processing
 (16) To be activated via rocker switch,
 Signia app or miniPocket

- (17) To be activated via push button,
 Signia app or miniPocket
 (18) Basic remote tuning via 4-band-equalizer
 (19) Full live remote tuning via Connexx
 (20) Not for 1px
 (21) With either CROS Pure 312 Nx or
 CROS Pure Charge&Go Nx
 (22) With CROS Pure

Overview BTEs

Motion SX/SA primax	Motion S primax	Intuis 3 S	Intuis 3 M	Intuis 3 P	Intuis 3 SP
7px/5px/3px/2px/1px	3px/2px/1px	Essential			
					
SX SA					
53/126 dB - 60/130 dB	- 45/124 dB 55/124 dB	- 45/124 dB 55/124 dB	- 53/125 dB 60/130 dB	- - 70/134 dB	- - 80/140 dB
13	312	312	13	13	675
1.0 mA	0.8 mA	0.9 mA	1.0 mA	1.4 mA	2.4 mA
7px - 48/20 5px - 32/16 3px - 24/12 2px - 16/8 ³⁰ 1px - 16/8 ³⁰	- - 3px - 24/12 2px - 16/8 1px - 16/8	12/6	12/6	12/6	12/6
6 ¹⁰ /4 ¹¹	6 ³¹ /4 ¹¹	4	4	4	4
● ¹⁴	● ¹⁴	-	-	-	-
-	-	-	-	-	-
● ^{2, 16}	-	-	-	-	-
● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸
● ²⁰	● ²⁰	-	-	-	-
● ^{2, 22}	-	-	-	-	-
-	● ²⁴	● ²⁴	-	●	●
● ²⁴	○ ²⁴	-	● ²⁴	● ²⁴	● ²⁴
● ²	-	-	-	-	-
●	-	-	●	●	●
○ ²⁹	○ ²⁹	-	○ ³³	○ ³³	○ ³³
○	○	○	○	○	-
●	●	●	●	●	●
●	●	-	-	-	-
-	-	-	-	-	-
●	●	●	●	●	●
●	●	-	-	-	-
●	●	-	-	-	-
-	-	-	-	-	-
size 13	size 312	size 312	size 13	Programming shoe/ CS 44 cable	Programming shoe/ CS 44 cable
ThinTube	LifeTube	LifeTube	LifeTube	-	-

(24) Programmable via Connexx

(28) Via direct streaming

(29) With Smart Connect

(30) For Motion SA primax only

(31) 6 hearing programs for 3px

(32) With battery compartment modification or Smart Connect

(33) With battery compartment modification

● Standard

○ Optional






Detailed feature overview **BTEs**.

Type	Prompt S	Prompt P	Prompt SP	Run S
Performance level	Lotus			
Picture				
Product details				
Matrix (2 ccm)	45/124 dB - 55/124 dB	- - 70/134 dB	- - 80/140 dB	45/124 dB - 55/124 dB
Battery size	312	13	675	312
Power consumption	0.9 mA	1.4 mA	2.4 mA	0.9 mA
Features				
Frequency channels/ Compression controls	8/4	8/4	8/4	8
Hearing programs	4	4	4	6 preconfigured sound profiles
e2e wireless	-	-	-	-
Direct streaming	-	-	-	-
Push button	●	●	●	-
Volume control	-	● ⁽²⁴⁾	● ⁽²⁴⁾	●
T-coil	-	●	●	-
Battery door on/off function	●	●	●	●
Audio input	-	○ ⁽³³⁾	○ ⁽³³⁾	-
App/Accessory compatibility				
Signia app	●	●	●	●
Smart Connect	-	-	-	-
miniPocket	●	●	●	●
Programming adapter/cable	size 312	Programming shoe/ CS 44 cable	Programming shoe/ CS 44 cable	MobileFitting app
Fitting set	LifeTube	Comfort acoustic coupling	Comfort acoustic coupling	Comfort acoustic coupling



(24) Programmable via Connexx
(33) With battery compartment modification

● Standard
○ Optional

Overview BTEs

Run P	Run SP	Fun P	Fun SP	Fast P
Lotus				
				
-	-	-	-	-
70/132 dB	79/140 dB	70/132 dB	79/140 dB	62/130 dB
13	675	13	675	13
1.3 mA	2.2 mA	1.2 mA	2.2 mA	1.1 mA
8	8	6	6	4
6 preconfigured sound profiles	6 preconfigured sound profiles	4 preconfigured sound profiles	4 preconfigured sound profiles	3 preconfigured sound profiles
-	-	-	-	-
-	-	-	-	-
-	-	-	-	-
●	●	●	●	●
-	-	-	-	-
●	●	●	●	●
-	-	-	-	-
●	●	-	-	-
-	-	-	-	-
●	●	-	-	-
MobileFitting app	MobileFitting app	Fitting via rocker switch	Fitting via rocker switch	Fitting via rocker switch
Comfort acoustic coupling	Comfort acoustic coupling	Comfort acoustic coupling	Comfort acoustic coupling	Simple acoustic coupling



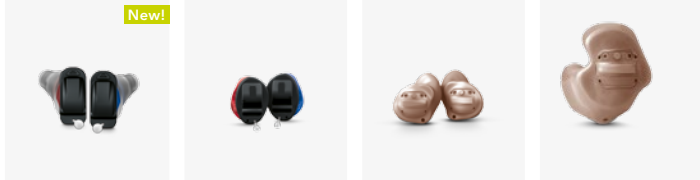




Detailed feature overview Click ITEs and ITEs.

Type	Silk Nx Click CIC	Insio Nx IIC/CIC	Insio Nx ITC	Insio Nx ITE	Silk primax Click CIC	Insio primax IIC/CIC
Performance levels	7Nx/5Nx/3Nx/2Nx/1Nx				7px/5px/3px/2px/1px	
Picture						
Product details						
Matrix (2 ccm)	50/113 dB	S 50/113 dB ¹ M 55/118 dB ¹ P 65/124 dB ¹	S 50/113 dB ¹ M 55/118 dB ¹ P 65/124 dB ¹	M 55/118 dB ¹ P 65/124 dB ¹	50/113 dB	65/124 dB ¹
Battery size	10	10	312	312	10	10
Power consumption	1.3 mA	1.3 mA ¹	1.3 mA ¹	1.3 mA ¹	1.1 mA	1.1 mA ¹
Features						
Frequency channels/ Compression controls	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 2Nx - 16/8 1Nx - 16/8	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 - -	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 - -	7Nx - 48/20 5Nx - 32/16 3Nx - 24/12 - -	7px - 48/20 5px - 32/16 3px - 24/12 2px - 16/8 1px - 16/8	7px - 48/20 5px - 32/16 3px - 24/12 2px - 16/8 1px - 16/8
Hearing programs	6	6	6	6	6 ^{10/4 11}	6 ^{10/4 11}
e2e wireless	● ¹³	○ ¹³	○ ¹³	○ ¹³	● ¹⁴	○ ¹⁴
Direct streaming	-	-	● ³⁴	● ³⁴	-	-
TwinPhone	● ¹⁵	● ³⁵	● ³⁵	● ³⁵	-	-
TeleCare	● ¹⁸	● ¹⁸	● ^{19, 34}	● ^{19, 34}	● ¹⁸	● ¹⁸
Tinnitus therapies	●	●	●	●	● ²⁰	● ²⁰
CROS/BiCROS	● ³⁶	● ³⁶	● ²¹	● ²¹	● ^{2, 22}	● ^{2, 22}
Push button	-	○ ³⁷	○ ³⁴	○ ³⁴	-	○ ³⁴
Volume control	○ ²⁵	○ ²⁵	○ ²⁵	○ ²⁵	○ ^{25, 29}	○ ^{25, 29}
T-coil	-	-	-	-	-	-
Battery door on/off function	●	●	●	●	●	●
Audio input	-	-	○ ²⁸	○ ²⁸	○ ²⁹	○ ²⁹
Optivent	-	○	○	○	-	○
App/Accessory compatibility						
Signia app	●	●	●	●	●	●
Smart Connect	-	-	-	-	●	●
StreamLine TV/StreamLine Mic	-	-	● ³⁴	● ³⁴	-	-
miniPocket	●	●	●	●	●	●
easyPocket	-	-	-	-	●	●
ConnexxAir	● ³⁸	● ³⁸	-	-	● ³⁸	● ³⁸
Noahlink Wireless	-	-	●	●	-	-
Programming adapter/cable	Flex connector	Flex connector ³⁷	Flex connector	Flex connector	Flex connector	Flex connector

- (1) Max. values
 (2) Not available with 2px/1px
 (10) 6 hearing programs for 7px/5px/3px
 (11) 4 hearing programs for 2px/1px
 (13) Ultra HD e2e
 with binaural audio signal processing
 (14) HD Binaural Hearing
 with binaural audio signal processing
 (15) To be activated via
 Signia app or miniPocket

- (18) Basic remote tuning via
 4-band-equalizer
 (19) Full live remote tuning via
 Connexx
 (20) Not for 1px
 (21) With CROS Pure 312 Nx
 (22) With CROS Pure
 (25) With Signia app/miniPocket
 (28) Via direct streaming
 (29) With Smart Connect
 (34) Only Bluetooth-enabled ITC/ITE

Overview Click ITEs/ITEs

Insio primax ITC	Insio primax ITE	Intuis 3 Click CIC	Intuis 3 Click ITC	Intuis 3 CIC	Intuis 3 ITC	Intuis 3 ITE
7px/5px/3px/2px/1px		Essential				
						
65/124 dB ¹	65/124 dB ¹	50/113 dB	55/116 dB	65/124 dB ¹	65/124 dB ¹	65/124 dB ¹
312/10	13/312	10	312	10	312	13
1.1 mA ¹	1.1 mA ¹	1.1 mA	1.1 mA	1.1 mA ¹	1.1 mA ¹	1.1 mA ¹
7px - 48/20 5px - 32/16 3px - 24/12 2px - 16/8 1px - 16/8	7px - 48/20 5px - 32/16 3px - 24/12 2px - 16/8 1px - 16/8	12/6	12/6	12/6	12/6	12/6
6 ¹⁰ /4 ¹¹	6 ¹⁰ /4 ¹¹	4	4	4	4	4
○ ¹⁴	○ ¹⁴	-	-	-	-	-
-	-	-	-	-	-	-
○ ²	○ ²	-	-	-	-	-
● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸	● ¹⁸
● ²⁰	● ²⁰	-	-	-	-	-
● ^{2, 22}	● ^{2, 22}	-	-	-	-	-
○	○	-	●	○	○	○
○ ^{25, 29}	○ ^{25, 29}	○ ²⁵	○ ²⁵	○ ²⁵	○ ²⁵	○ ²⁵
○	○	-	-	-	○	○
●	●	●	●	●	●	●
○ ²⁹	○ ²⁹	-	-	-	-	-
○	○	-	-	○	○	○
●	●	●	●	●	●	●
●	●	-	-	-	-	-
-	-	-	-	-	-	-
●	●	●	●	●	●	●
●	●	-	-	-	-	-
● ³⁸	● ³⁸	-	-	-	-	-
-	-	-	-	-	-	-
Flex connector	Flex connector	Flex connector	Flex connector	Flex connector	Flex connector	Flex connector

(35) To be activated via Signia app, miniPocket or optional push button






(36) With CROS Silk Nx

(37) Not available for IIC

(38) Only if e2e wireless is available

● Standard
○ Optional

Detailed feature overview **Click ITEs** and **ITEs**.

Type	Prompt Click CIC	Prompt Click ITC	Prompt CIC	Prompt ITC	Prompt ITE
Performance levels	Lotus				
Picture					
Product details					
Matrix (2 ccm)	50/113 dB	55/116 dB	65/124 dB ¹	65/124 dB ¹	65/124 dB ¹
Battery size	10	312	10	312/10	13/312
Power consumption	1.1 mA	1.1 mA	1.1 mA ¹	1.1 mA ¹	1.1 mA ¹
Features					
Frequency channels/ Compression controls	8/4	8/4	8/4	8/4	8/4
Hearing programs	4	4	4	4	4
Push button	-	●	○	○	○
Volume control	-	-	○ ²⁵	○ ²⁵	○ ²⁵
T-coil	-	-	-	○	○
Battery door on/off function	●	●	●	●	●
Optivent	-	-	○	○	○
App/Accessory compatibility					
Signia app	●	●	●	●	●
miniPocket	●	●	●	●	●
Programming adapter/cable	Flex connector	Flex connector	Flex connector	Flex connector	Flex connector



(1) Max. values

(25) With Signia app/miniPocket

● Standard

○ Optional

Overview Click ITEs/ITEs

Run Click CIC	Run Click ITC
Lotus	
	
50/113 dB	55/116 dB
10	312
1.1 mA	1.1 mA
8	8
6 preconfigured sound profiles	6 preconfigured sound profiles
-	-
-	-
-	-
●	●
-	-
●	●
●	●
MobileFitting app	MobileFitting app

The information in this document contains general descriptions of the technical options available, which do not always have to be present in individual cases and are subject to change without prior notice.

The Bluetooth word mark and logos are owned by the Bluetooth SIG, Inc. and any use of such marks by Sivantos is under license. Other trademarks and trade names are those of their respective owners.

* Study conducted at the University of Northern Colorado, 2015, examined the effectiveness of the new features of primax by collecting and analyzing ongoing EEG data while subjects performed speech testing. For both primax features SpeechMaster and EchoShield, the objective brain behavior measures revealed a significant reduction in listening effort when the feature was activated.

Legal Manufacturer

Signia GmbH
Henri-Dunant-Strasse 100
91058 Erlangen
Germany

Order No. 03984-99C1-7600 | Printed in Germany
©08.2019 Signia GmbH

